


00VEN-0074-01

Nasal Tip Replacement


1: Loosen friction knob and unsnap the boom arm from the ball mount




2: Slide the vinyl retaining ring off towards the cannula tip and set aside. Note the position of the arm for easier repositioning.

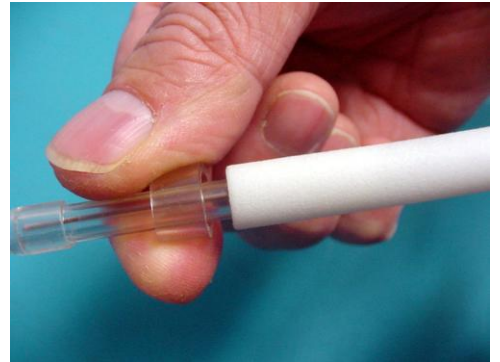

3: Snap the cannula assembly out of the boom arm and slide back through the slot. It will be a tight fit.

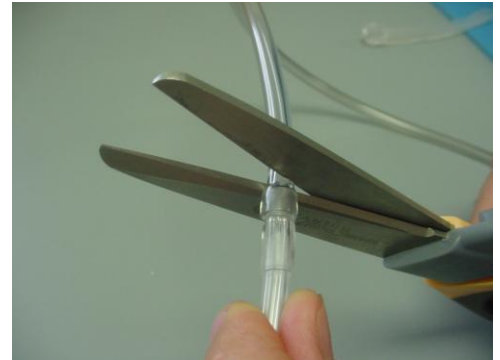


4: Cut the vinyl tubing off close to the adaptor sleeve. Discard the old cannula.

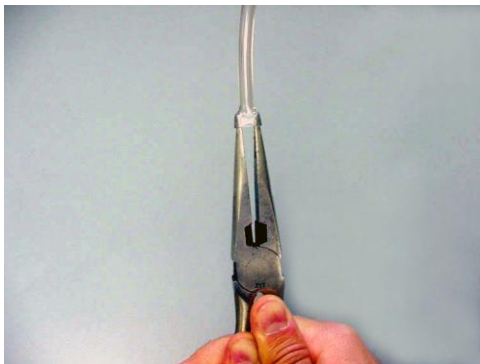

5: The cut end of the vinyl tubing will now have to be stretched to fit over the new adapter sleeve. This can be done by immersing the cut tube end in HOT water, inserting clean needle nose pliers and pulling the handles apart.

6: Work quickly to put the stretched tube end over the reducing adapter.

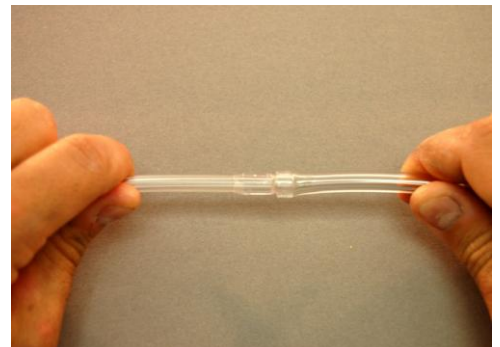

7: Reassemble cannula in reverse order and fly away happy.