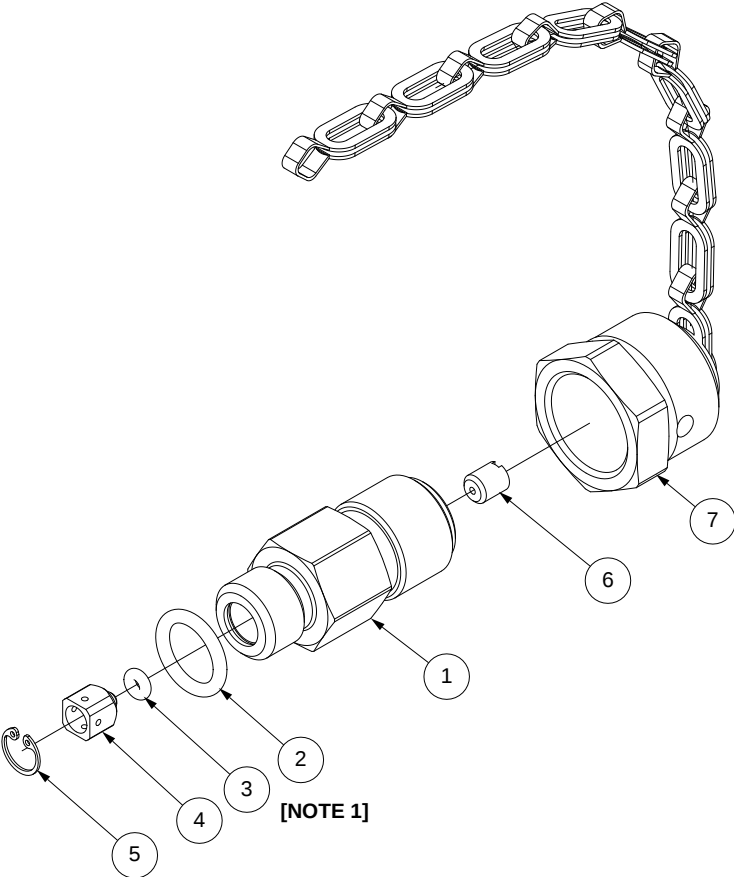
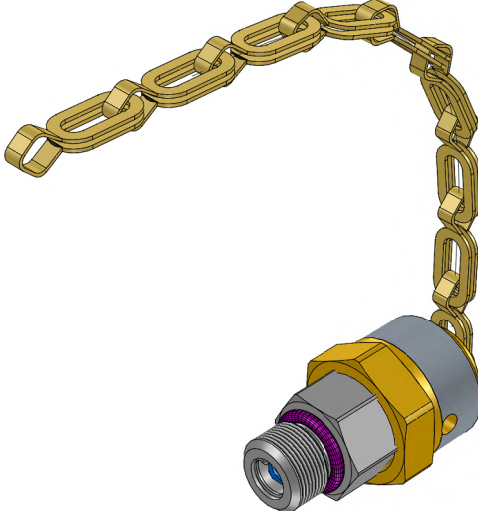


1		2		3		4																																									
A		REVISION HISTORY <table><tr><th>REV</th><th>ECO NO. YYYY-MM-DD</th><th>NAME</th><th>NOTES</th></tr><tr><td>D0</td><td>2015-006 2015-02-23</td><td>SGO</td><td>Change Item 3 O-ring: was 09001-1003-70 (Silicon), is 09001-3003-70 (EPDM)</td></tr></table>						REV	ECO NO. YYYY-MM-DD	NAME	NOTES	D0	2015-006 2015-02-23	SGO	Change Item 3 O-ring: was 09001-1003-70 (Silicon), is 09001-3003-70 (EPDM)	A																															
		REV	ECO NO. YYYY-MM-DD	NAME	NOTES																																										
		D0	2015-006 2015-02-23	SGO	Change Item 3 O-ring: was 09001-1003-70 (Silicon), is 09001-3003-70 (EPDM)																																										
																																															
NOTES: [1] DO NOT LUBE Item 3 O-ring																																															
D		<table><tr><th>REF</th><th>QTY</th><th>Part Number</th><th>Rev</th><th>Description</th></tr><tr><td>1</td><td>1</td><td>OCV04-0201-00</td><td>A0</td><td>Check-Valve Body, Poppet Style, Scott 9/16-18 x SAE-4M</td></tr><tr><td>2</td><td>1</td><td>09001-3904-70</td><td></td><td>O-ring, EPDM, 3-904 (SAE-4), E70</td></tr><tr><td>3</td><td>1</td><td>09001-3003-70</td><td></td><td>O-ring, EPDM, 2-003, E70</td></tr><tr><td>4</td><td>1</td><td>OCV02-0201-00</td><td>A</td><td>Poppet, 0.25 Dia.</td></tr><tr><td>5</td><td>1</td><td>09007-0001-00</td><td></td><td>Retaining Ring, 1/4" Internal, SS) [Rotor Clip HO-25SS MP...</td></tr><tr><td>6</td><td>1</td><td>09150-0803-05</td><td></td><td>Threaded Filter Insert, #8-32 x .190) [Bird RB-91097-04449-...</td></tr><tr><td>7</td><td>1</td><td>OCP02-0103-01</td><td>-0</td><td>Cap & Chain Assembly, Scott 9/16-18</td></tr></table>						REF	QTY	Part Number	Rev	Description	1	1	OCV04-0201-00	A0	Check-Valve Body, Poppet Style, Scott 9/16-18 x SAE-4M	2	1	09001-3904-70		O-ring, EPDM, 3-904 (SAE-4), E70	3	1	09001-3003-70		O-ring, EPDM, 2-003, E70	4	1	OCV02-0201-00	A	Poppet, 0.25 Dia.	5	1	09007-0001-00		Retaining Ring, 1/4" Internal, SS) [Rotor Clip HO-25SS MP...	6	1	09150-0803-05		Threaded Filter Insert, #8-32 x .190) [Bird RB-91097-04449-...	7	1	OCP02-0103-01	-0	Cap & Chain Assembly, Scott 9/16-18
		REF	QTY	Part Number	Rev	Description																																									
		1	1	OCV04-0201-00	A0	Check-Valve Body, Poppet Style, Scott 9/16-18 x SAE-4M																																									
		2	1	09001-3904-70		O-ring, EPDM, 3-904 (SAE-4), E70																																									
		3	1	09001-3003-70		O-ring, EPDM, 2-003, E70																																									
4	1	OCV02-0201-00	A	Poppet, 0.25 Dia.																																											
5	1	09007-0001-00		Retaining Ring, 1/4" Internal, SS) [Rotor Clip HO-25SS MP...																																											
6	1	09150-0803-05		Threaded Filter Insert, #8-32 x .190) [Bird RB-91097-04449-...																																											
7	1	OCP02-0103-01	-0	Cap & Chain Assembly, Scott 9/16-18																																											
1				2																																											
3				4																																											